



TeMeCo B.V.

The Netherlands

www.temeco.nl

info@temeco.nl

SenseLink
The new experience in ECG



Application Instructions

3M RED DOT ELECTRODES FOR

SenseLink
The new experience in ECG

3M RED DOT ELECTRODES FOR SENSELINK

Application Instructions



STEP 1: CHECK YOUR ELECTRODES

- ✓ Use the 3M Red Dot 2255-50 electrodes
- ✓ Do not use outdated electrodes
- ✓ Do not use electrodes 30 days after opening the package

STEP 2: CLEAN THE SKIN

- ✓ The skin must be clean, dry and free of any body lotions
- ✓ Clip excessive hair
- ✓ Wash site with soap and water
- ✓ Avoid using alcohol



STEP 3: PREPARE THE SKIN

- ✓ Abrade the skin by marking an "X" on the skin
- ✓ Use the abradant pad on the back side of the electrode



STEP 4: APPLY THE ELECTRODES

- ✓ Avoid pressing on the metal stud when the electrode is applied on the patient
- ✓ Connect the snaps to the electrode prior to placing it on the patient
- ✓ Firmly rub your finger around the outside of the front label to improve adhesion



STEP 5: REMOVE THE ELECTRODES

- ✓ Loosen one side of the electrode
- ✓ Grasp the full width of the electrode, slowly and gently pull it back over itself
- ✓ Electrodes should be removed in the direction of hair growth (whenever possible)
- ✓ Keep the electrode close to the skin surface during removal